


Flip for x:panda Caster Arm

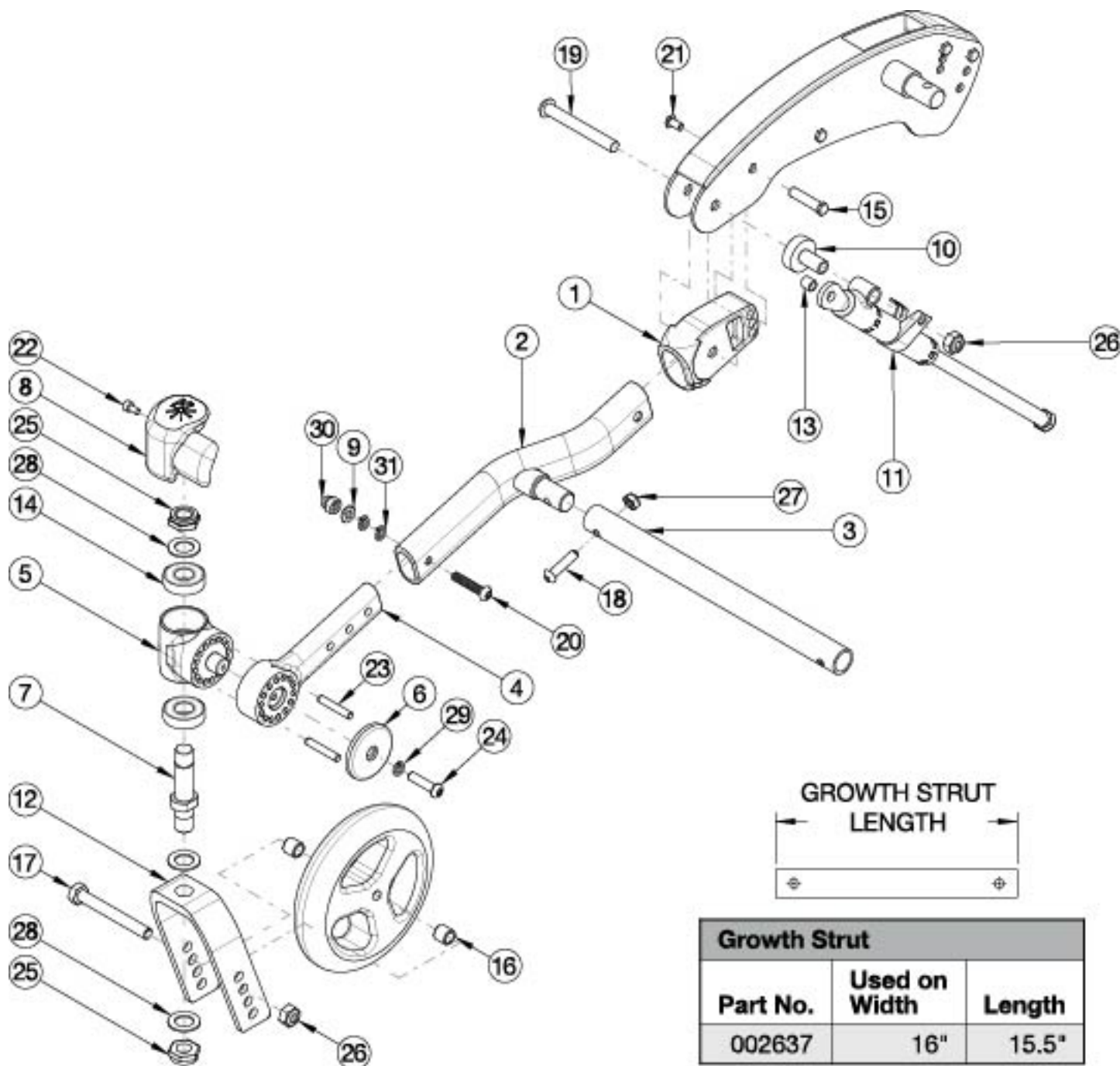

If chair was invoiced prior to 5/1/2024, the Lower Caster Arm Assembly for both sides is required when replacement caster barrel, caster arm, or caster arm cover are requested.

Item Number	Part Number	Description	Quantity
10, 11a, 13, 19, 26	503213	Mechlock Replacement Kit, 45° Right	1
10, 11b, 13, 19, 26	503214	Mechlock Replacement Kit, 45° Left	1

Item Number	Part Number	Description	Quantity
10, 11a, 11b, 13, 19, 26	502797	Mechlock Replacement Kit, 45° PAIR FLIP	1
16a, 17a, 26	502653	Caster Attachment Hardware Kit CAT/FCS	1
16b, 17b, 26	502654	Caster Attachment Hardware Kit for 6" and 7" Wide Fork, CAT/FCS	1
5, 14	502801	Caster Barrel with bearings FLIP	1
7, 12a, 25, 28	502803	Fork Assembly 6" FLIP	1
7, 12b, 25, 28	502804	Fork Assembly 6" Wide FLIP	1
7, 12c, 25, 28	502805	Fork Assembly 7" FLIP	1
1, 2, 4-6, 8a, 9, 14, 15, 18-24, 26, 27, 29-31	502799_ _ _	Caster Arm Assembly, Right	1
1, 2, 4-6, 8b, 9, 14, 15, 18-24, 26, 27, 29-31	502800_ _ _	Caster Arm Assembly, Left	1
4-6, 8a, 14, 22- 24, 29	113927	Lower Caster Arm Assembly, Right	1
4-6, 8b, 14, 22- 24, 29	113928	Lower Caster Arm Assembly, Left	1
1, 2	503673_ _ _	Front Frame Tube Weldment w/Adjuster, Flip	1
3	002637	Frame Growth Strut 15.5"	1
4	122586	Caster Arm	1
6	122588	Caster Arm Cover	1
7	501208	Caster Stem, Ped Tilt	1
8a	002651	Wear Pad, Caster, Right	1
8b	002652	Wear Pad, Caster, Left	1
9	100746	M6 Flat Washer Blk Zn <i>Not used if transit is present</i>	1
10	001865	Spacer, Mechlock	1
11a	001861	Mechlock, 158mm, Right	1
11b	001862	Mechlock, 158mm, Left	1
12a	501210	Caster Fork 6" Flip	1
12b	501211	Caster Fork 6" Wide Flip	1
12c	501212	Caster Fork 7" Flip	1
12d	501213	Caster Fork 7" Wide Flip	1

Item Number	Part Number	Description	Quantity
13	001156	Pivot Bushing Tilt	2
15	000987	Threaded Fastener	1
16a	100792	Caster Spacer D8 x D12 x 10mm / Close Mount Handrim Spacer <i>Used with all casters except 6x2</i>	2
16b	001246	Caster Spacer Black 6 x 2 Poly .472ODX.319IDX.370LG <i>Only used with 6" x 2" casters</i>	2
17a	101829	M8 x 65 HHCS Blk Zn	1
17b	101831	M8 x 80 HHCS Blk Zn <i>Used with wide forks</i>	1
18	101593	M6 x 25 BTN SHCS BLKZC	1
19	102797	M8 x 65, BHCS, Blk Zn	1
20a	102625	M6 X 30 BHCS BLK ZC <i>Used if transit is not present</i>	1
20b	101733	M6 x 35 CLS10.9 DIN7991 FSHCS BLKZC <i>Used if transit is present</i>	1
21	100669	M5 x 10 BTN SHCS BLKZC w/Patch	1
22	102800	M4 x 8mm SHCS Blk Zn	1
23	122589	Pin, Dowel, M5 X 30, DIN6325	2
24	115573	M6 x 25, BHCS, CLS 10.9, NYL, BLZ	1
25	100529	M12 x 1.25mm Lock Nut Black	2
26	100659	M8 Nylock Nut Blk Zn	2
27	100658	M6 Nylock Nut Blk Zn	1
28	100682	Washer 1/2" .518 x .875 x .047 F/W Black Zinc	3
29	102614	M5 Split Lock Washer BLK ZC	1
30	100558	M6 Nyl Ins Domed Cap L/N Zc	1
31	102810	M6 Lockwasher, Hi-Collar, ZC <i>Not used if transit is present</i>	2
	002086	Spacer, .38ODx.24IDx.31LG <i>Used if transit is present - replaces callouts 9 and 31. See Flip for x:panda Transit page.</i>	1