



ALLCOURT / MATCH POINT RATCHET BELT




Please read these instructions carefully before beginning the assembly. Failure to understand and follow assembly instructions may result in injury to technician and/or end user and may void the warranty. If you have any questions call Sunrise Medical Technical Support at +1 (800) 333-4000.

ITEM NO.	QTY	DESCRIPTION
1	2	1/2-IN SCREW
2	2	NYLOCK NUT
3	2	RATCHET POSITIONING BELT BRACKET
4	1	RATCHET POSITIONING BELT

The following assembly instructions are for the ratchet positioning belt for the RGK AllCourt and Match Point sport-chairs.

To install ratchet positioning belt:

1. Attach ratchet positioning belt bracket (item 3) to the top of the seat sling using the existing screws that came with the chair.
2. Attach the belt to the bracket with the nut and screw provided (items 1 and 2). The belt can be attached to either the inside or outside of the bracket.
3. Repeat on the other side of the chair.

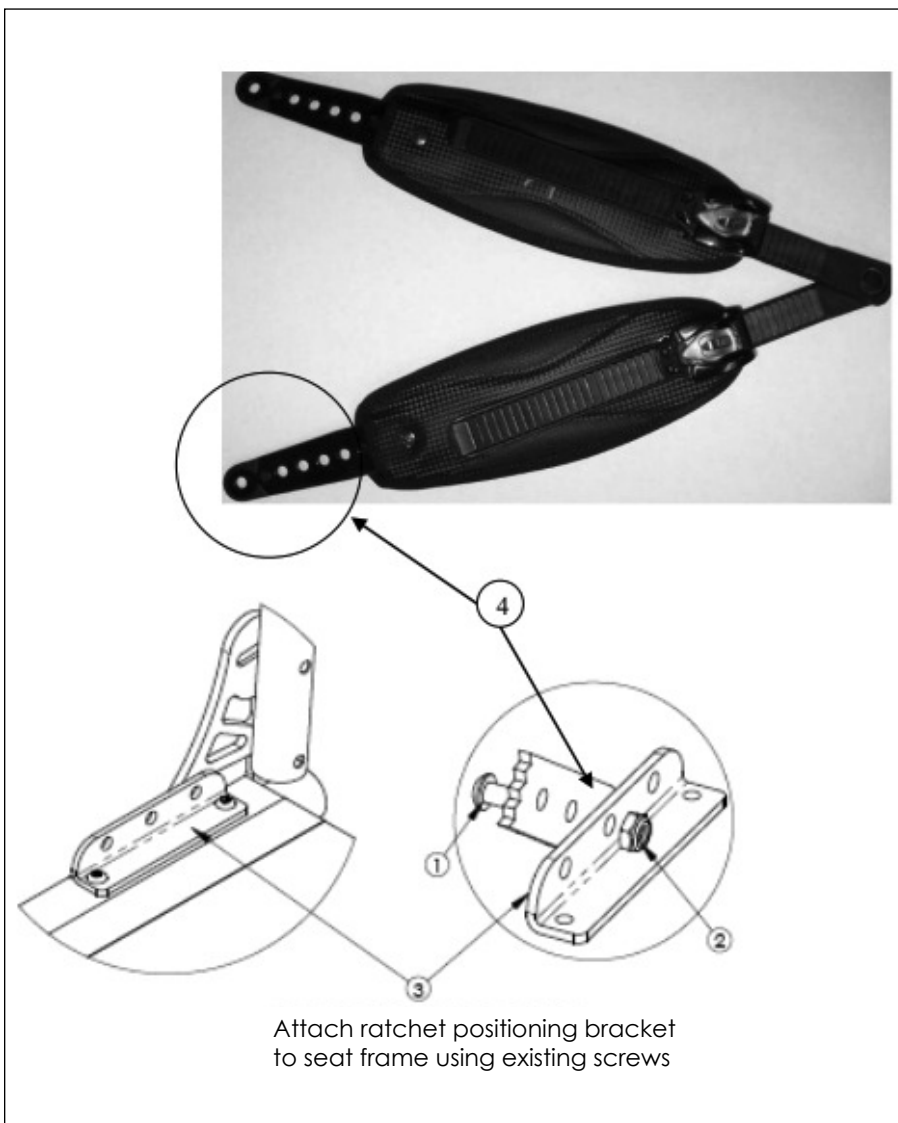

⚠️ DEALER/TECHNICIAN WARNING

Attention dealers and qualified technicians, do not operate or service this device without first reading the owner's manual. If you do not understand the instructions and warnings in the owner's manual please contact the Sunrise Technical Service Department before operating and/or servicing the RGK wheelchair. Failure to do so may result in damage and/or injury.

Find more information and important warnings in the wheelchair owner's manual or at: www.SunriseMedical.com



Customer Service: +1 (800) 333-4000
In Canada (800) 263-3390
or visit www.SunriseMedical.com

© 2023 Sunrise Medical (US) LLC
07/2023 P/N 950157 Rev. D
Pg 1 of 1