


# Catalyst E Seat Upholstery

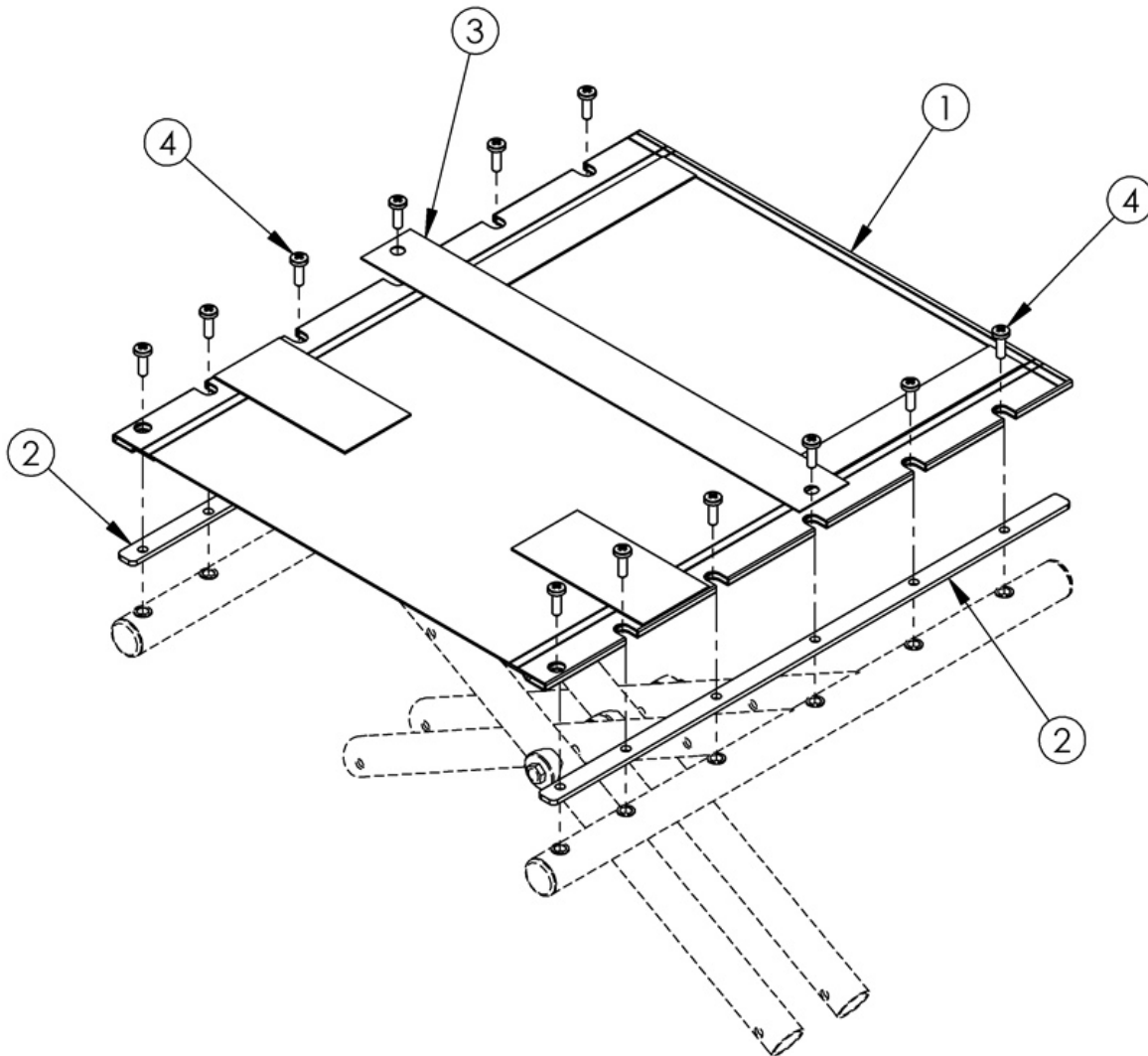

SCREW (ITEM 4) QTY PER CHAIR	
SEAT DEPTH	SCREW QTY
14" - 15"	10
16" - 17"	12
18" - 19"	14
20"	16

*Seat upholstery width measures approximately 2.5" less than the chair size. Seat upholstery depth measures true to the chair size.*

Item Number	Part Number	Description	Quantity
1a	100095	Seat Upholstery Cat 1416	1

<b>Item Number</b>	<b>Part Number</b>	<b>Description</b>	<b>Quantity</b>
1b	100097	Seat Upholstery Cat 1418	1
1c	100100	Seat Upholstery Cat 1516	1
1d	100102	Seat Upholstery Cat 1518	1
1e	100104	Seat Upholstery Cat 1520	1
1f	100107	Seat Upholstery Cat 1616	1
1g	100109	Seat Upholstery Cat 1618	1
1h	100111	Seat Upholstery Cat 1620	1
1i	100114	Seat Upholstery Cat 1716	1
1j	100116	Seat Upholstery Cat 1718	1
1k	100118	Seat Upholstery Cat 1720	1
1l	100121	Seat Upholstery Cat 1816	1
1m	100123	Seat Upholstery Cat 1818	1
1n	100125	Seat Upholstery Cat 1820	1
1o	100126	Seat Upholstery Cat 1916	1
1p	100128	Seat Upholstery Cat 1918	1
1q	100130	Seat Upholstery Cat 1920	1
1r	100131	Seat Upholstery Cat 2016	1
1s	100133	Seat Upholstery Cat 2018	1
1t	100135	Seat Upholstery Cat 2020	1
1u	101450	Seat Upholstery Cat 2116	1
1v	100855	Seat Upholstery Cat 2118	1
1w	100857	Seat Upholstery Cat 2120	1
1x	101556	Seat Upholstery Cat 2216	1
1y	100247	Seat Upholstery Cat 2218	1
1z	100261	Seat Upholstery Cat 2220	1
2a	100232	Seat Rail 16" Deep CAT	2
2b	100234	Seat Rail 18" Deep CAT	2
2c	100236	Seat Rail 20" Deep CAT	2

<b>Item Number</b>	<b>Part Number</b>	<b>Description</b>	<b>Quantity</b>
3a	000059	Lifting Strap 14" Wide CAT	1
3b	000060	Lifting Strap 15" Wide CAT	1
3c	000061	Lifting Strap 16" Wide CAT	1
3d	000062	Lifting Strap 17" Wide CAT	1
3e	000063	Lifting Strap 18" Wide CAT	1
3f	000064	Lifting Strap 19" Wide CAT	1
3g	000058	Lifting Strap 20" Wide CAT	1
3h	100248	Lifting Strap 21" Wide CAT	1
3i	100875	Lifting Strap 22" Wide CAT	1
4	100734	M5 x 16 PHMS ZC	16