


Rogue XP 1" Offset Backrest Mount

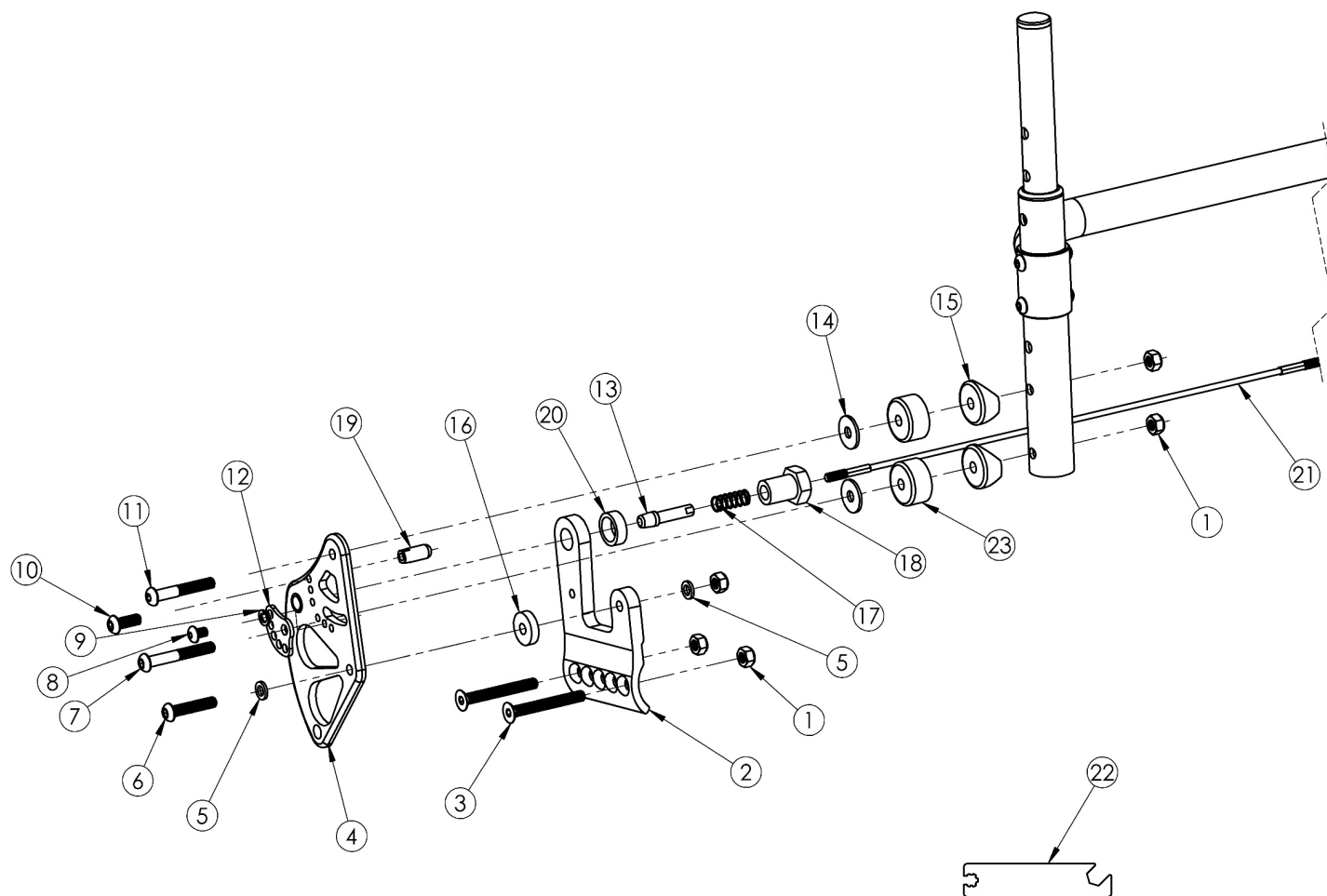

1" Offset Backrest Mount used with Height Adjustable Back Posts

<p>

Spinergy Back Release Cable based on Seat Width

Item Number	Part Number	Description	Quantity	Retail
1, 2a, 3a - 3c, 4a, 5-20, 23	503698	1" Inward Offset Backrest Mount Assembly, Right, Black, Rogue XP	1	\$167.11
1, 2a, 3a - 3c, 4b, 5-20, 23	503698_ _ _	1" Inward Offset Backrest Mount Assembly, Right, Color Anodized, Rogue XP	1	\$208.74
1, 2b, 3a - 3c, 4c, 5-20, 23	503699	1" Inward Offset Backrest Mount Assembly, Left, Black, Rogue XP	1	\$167.11
1, 2b, 3a - 3c, 4d, 5-20, 23	503699_ _ _	1" Inward Offset Backrest Mount Assembly, Left, Color Anodized, Rogue XP	1	\$208.74
1, 2a, 2b, 3a - 3c, 4a, 4c, 5-20, 23	503700	1" Inward Offset Backrest Mount Assembly, PAIR, Black, Rogue XP	1	\$334.21
1, 2a, 2b, 3a - 3c, 4b, 4d, 5-20, 23	503700_ _ _	1" Inward Offset Backrest Mount Assembly, PAIR, Color Anodized, Rogue XP	1	\$417.49
1	100658	M6 Nylock Nut Blk Zn	5	\$1.05

Item Number	Part Number	Description	Quantity	Retail
2a	003098	XP Back Mount Right Rogue	1	\$28.37
2b	003099	XP Back Mount Left Rogue	1	\$28.37
3a	101129	M6 x 50 FSHCS P/THD BLKZC <i>Used when transit is not present</i>	2	\$2.10
3b	103068	M6 x 60mm FHCS Blk Zn <i>Use quantity 2 per side when transit is present</i>	2	\$2.10
3c	101130	M6 x 55 FSHCS P/THD BLKZC <i>Use quantity 1 per side when transit is present (not shown in image)</i>	1	\$2.10
4a	003229	Back Bracket Relaxed Assembly Right, Black Anodized	1	\$63.04
4b	002770_ _ _	Back Bracket Relaxed Right Color Anodized	1	\$104.67
4c	003231	Back Bracket Relaxed Assembly Left, Black Anodized	1	\$63.04
4d	002771_ _ _	Back Bracket Relaxed Left Color Anodized	1	\$104.67
5	102072	Washer Nylon .234 ID x .406 OD x .062 Thick	2	\$0.53
6	102625	M6 X 30 BHCS BLK ZC	1	\$1.58
7	003241	M6 x 60 BHCS P/THD Custom Blk Zc	1	\$11.56
8	102621	M5 x 8 BHCS BLK ZC	1	\$0.53
9	102614	M5 Split Lock Washer BLK ZC	1	\$0.53
10	101716	M5 x 16 SHCS Blk Zn	1	\$1.05
11	003232	M6 x 55 BHCS Blk Zc	1	\$8.94
12	001762	Arc Adjustment Cam	1	\$3.98
13	000909	Backrest Lockout Pin RG	1	\$15.63
14	102629	Seastrom 5610-69-062 Washer Nylon Black	2	\$1.05
15	000055	Saddle 1"	2	\$1.58
16	102630	Washer Nylon 25 X 75 X .187 Black	1	\$1.05
17	000965	Backrest Cable Release Spring RG	1	\$1.79
18	001768	Pin Receiver with Hex	1	\$5.47
19	000922	Backrest Stop RG	1	\$5.01
20	001770	Spacer For Pin Receiver <i>This spacer is not used if Swing Away Armrests are present</i>	1	\$3.19

Item Number	Part Number	Description	Quantity	Retail
21a	102572	Spinergy Back Release Cable 10" Wide Black	1	\$31.52
21b	102573	Spinergy Back Release Cable 10" Wide Blue	1	\$31.52
21c	102574	Spinergy Back Release Cable 10" Wide Red	1	\$31.52
21d	102575	Spinergy Back Release Cable 10" Wide Yellow	1	\$31.52
21e	102576	Spinergy Back Release Cable 10" Wide Green	1	\$31.52
21f	102577	Spinergy Back Release Cable 10" Wide Pink	1	\$31.52
21g	102578	Spinergy Back Release Cable 10" Wide Orange	1	\$31.52
21h	102579	Spinergy Back Release Cable 10" Wide White	1	\$31.52
21q	102580	Spinergy Back Release Cable 11" Wide Black	1	\$31.52
21r	102581	Spinergy Back Release Cable 11" Wide Blue	1	\$31.52
21s	102582	Spinergy Back Release Cable 11" Wide Red	1	\$31.52
21t	102583	Spinergy Back Release Cable 11" Wide Yellow	1	\$31.52
21u	102584	Spinergy Back Release Cable 11" Wide Green	1	\$31.52
21v	102585	Spinergy Back Release Cable 11" Wide Pink	1	\$31.52
21w	102586	Spinergy Back Release Cable 11" Wide Orange	1	\$31.52
21x	102587	Spinergy Back Release Cable 11" Wide White	1	\$31.52
21ag	200260	Spinergy Back Release Cable 12" Wide Black	1	\$31.52
21ah	200261	Spinergy Back Release Cable 12" Wide Blue	1	\$31.52
21ai	200262	Spinergy Back Release Cable 12" Wide Red	1	\$31.52
21aj	200263	Spinergy Back Release Cable 12" Wide Yellow	1	\$31.52
21ak	200264	Spinergy Back Release Cable 12" Wide Green	1	\$31.52
21al	200265	Spinergy Back Release Cable 12" Wide Pink	1	\$31.52
21am	200266	Spinergy Back Release Cable 12" Wide Orange	1	\$31.52
21an	200267	Spinergy Back Release Cable 12" Wide White	1	\$31.52
21aw	200268	Spinergy Back Release Cable 13" Wide Black	1	\$31.52
21ax	200269	Spinergy Back Release Cable 13" Wide Blue	1	\$31.52
21ay	200270	Spinergy Back Release Cable 13" Wide Red	1	\$31.52
21az	200271	Spinergy Back Release Cable 13" Wide Yellow	1	\$31.52

Item Number	Part Number	Description	Quantity	Retail
21ba	200272	Spinergy Back Release Cable 13" Wide Green	1	\$31.52
21bb	200273	Spinergy Back Release Cable 13" Wide Pink	1	\$31.52
21bc	200274	Spinergy Back Release Cable 13" Wide Orange	1	\$31.52
21bd	200275	Spinergy Back Release Cable 13" Wide White	1	\$31.52
21bm	200276	Spinergy Back Release Cable 14" Wide Black	1	\$31.52
21bn	200277	Spinergy Back Release Cable 14" Wide Blue	1	\$31.52
21bo	200278	Spinergy Back Release Cable 14" Wide Red	1	\$31.52
21bp	200279	Spinergy Back Release Cable 14" Wide Yellow	1	\$31.52
21bq	200280	Spinergy Back Release Cable 14" Wide Green	1	\$31.52
21br	200281	Spinergy Back Release Cable 14" Wide Pink	1	\$31.52
21bs	200282	Spinergy Back Release Cable 14" Wide Orange	1	\$31.52
21bt	200283	Spinergy Back Release Cable 14" Wide White	1	\$31.52
21cc	200284	Spinergy Back Release Cable 15" Wide Black	1	\$31.52
21cd	200285	Spinergy Back Release Cable 15" Wide Blue	1	\$31.52
21ce	200286	Spinergy Back Release Cable 15" Wide Red	1	\$31.52
21cf	200287	Spinergy Back Release Cable 15" Wide Yellow	1	\$31.52
21cg	200288	Spinergy Back Release Cable 15" Wide Green	1	\$31.52
21ch	200289	Spinergy Back Release Cable 15" Wide Pink	1	\$31.52
21ci	200290	Spinergy Back Release Cable 15" Wide Orange	1	\$31.52
21cj	200291	Spinergy Back Release Cable 15" Wide White	1	\$31.52
21cs	200292	Spinergy Back Release Cable 16" Wide Black	1	\$31.52
21ct	200293	Spinergy Back Release Cable 16" Wide Blue	1	\$31.52
21cu	200294	Spinergy Back Release Cable 16" Wide Red	1	\$31.52
21cv	200295	Spinergy Back Release Cable 16" Wide Yellow	1	\$31.52
21cw	200296	Spinergy Back Release Cable 16" Wide Green	1	\$31.52
21cx	200297	Spinergy Back Release Cable 16" Wide Pink	1	\$31.52
21cy	200298	Spinergy Back Release Cable 16" Wide Orange	1	\$31.52
21cz	200299	Spinergy Back Release Cable 16" Wide White	1	\$31.52

Item Number	Part Number	Description	Quantity	Retail
21di	200300	Spinergy Back Release Cable 17" Wide Black	1	\$31.52
21dj	200301	Spinergy Back Release Cable 17" Wide Blue	1	\$31.52
21dk	200302	Spinergy Back Release Cable 17" Wide Red	1	\$31.52
21dl	200303	Spinergy Back Release Cable 17" Wide Yellow	1	\$31.52
21dm	200304	Spinergy Back Release Cable 17" Wide Green	1	\$31.52
21dn	200305	Spinergy Back Release Cable 17" Wide Pink	1	\$31.52
21do	200306	Spinergy Back Release Cable 17" Wide Orange	1	\$31.52
21dp	200307	Spinergy Back Release Cable 17" Wide White	1	\$31.52
21dy	200308	Spinergy Back Release Cable 18" Wide Black	1	\$31.52
21dz	200309	Spinergy Back Release Cable 18" Wide Blue	1	\$31.52
21ea	200310	Spinergy Back Release Cable 18" Wide Red	1	\$31.52
21eb	200311	Spinergy Back Release Cable 18" Wide Yellow	1	\$31.52
21ec	200312	Spinergy Back Release Cable 18" Wide Green	1	\$31.52
21ed	200313	Spinergy Back Release Cable 18" Wide Pink	1	\$31.52
21ee	200314	Spinergy Back Release Cable 18" Wide Orange	1	\$31.52
21ef	200315	Spinergy Back Release Cable 18" Wide White	1	\$31.52
22	101991	Spinergy Release Cable/SPOX Adjustment Tool <i>No charge when ordered with Spinergy Release Cable</i>	1	
23	002475	Offset Backrest Spacer	2	\$7.75

Color Anodizing Samples

