


Rogue XP Transit

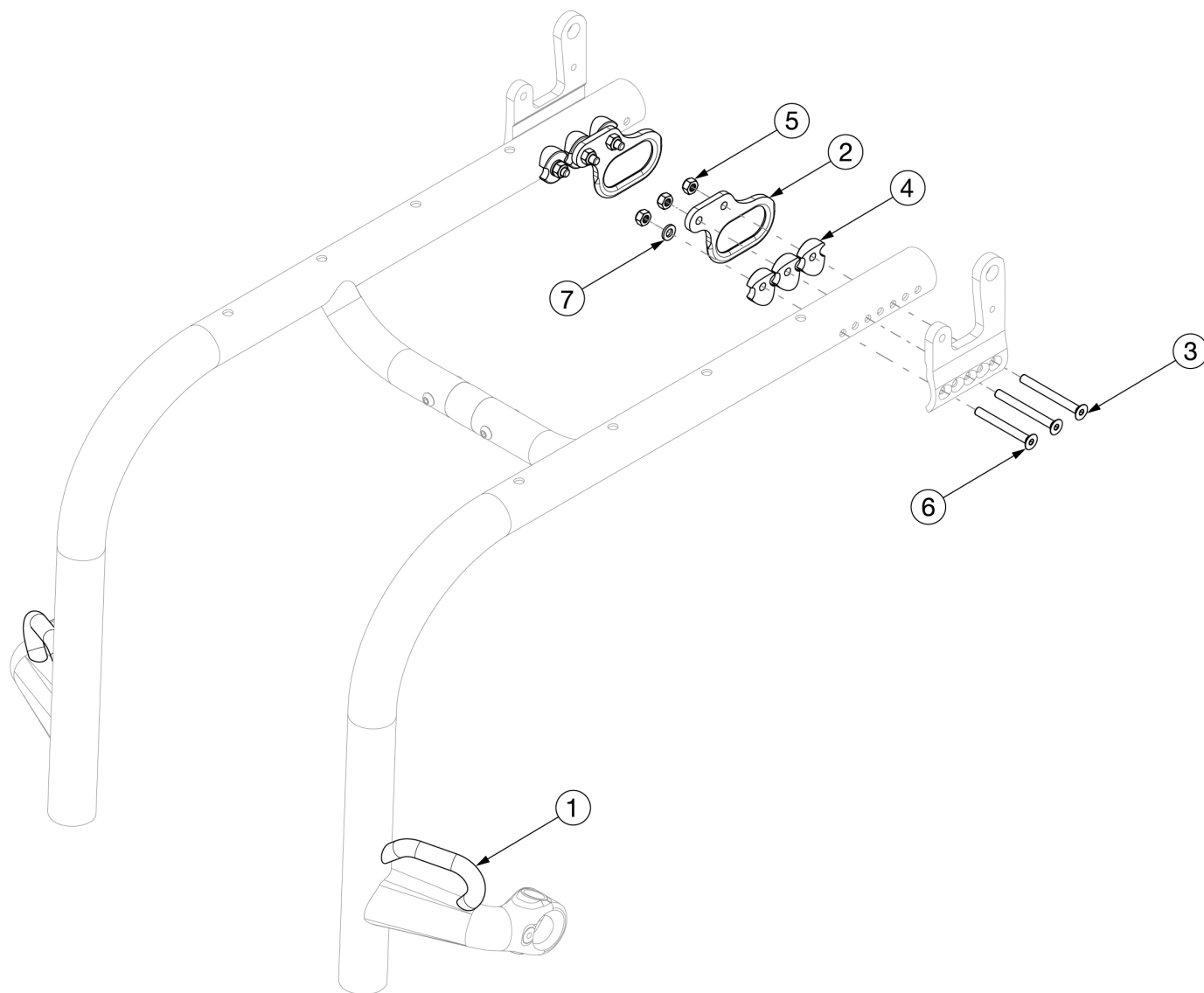

Rogue XP Transit System cannot be retrofitted. If needing to add transit, please contact Customer Service. Used on chairs invoiced starting on 3/12/25

Item Number	Part Number	Description	Quantity
2-7	124937	Rear Transit Brackets with Hardware	1
2	001782	Rear Transit Bracket	2
3	103068	M6 x 60mm FHCS Blk Zn	4
4	000248	Saddle 1.375"	6
5	100658	M6 Nylock Nut Blk Zn	6

Item Number	Part Number	Description	Quantity
6	101130	M6 x 55 FSHCS P/THD BLKZC	2
7	100746	M6 Flat Washer Blk Zn	2