


Axiom for Kids DEX Hardware

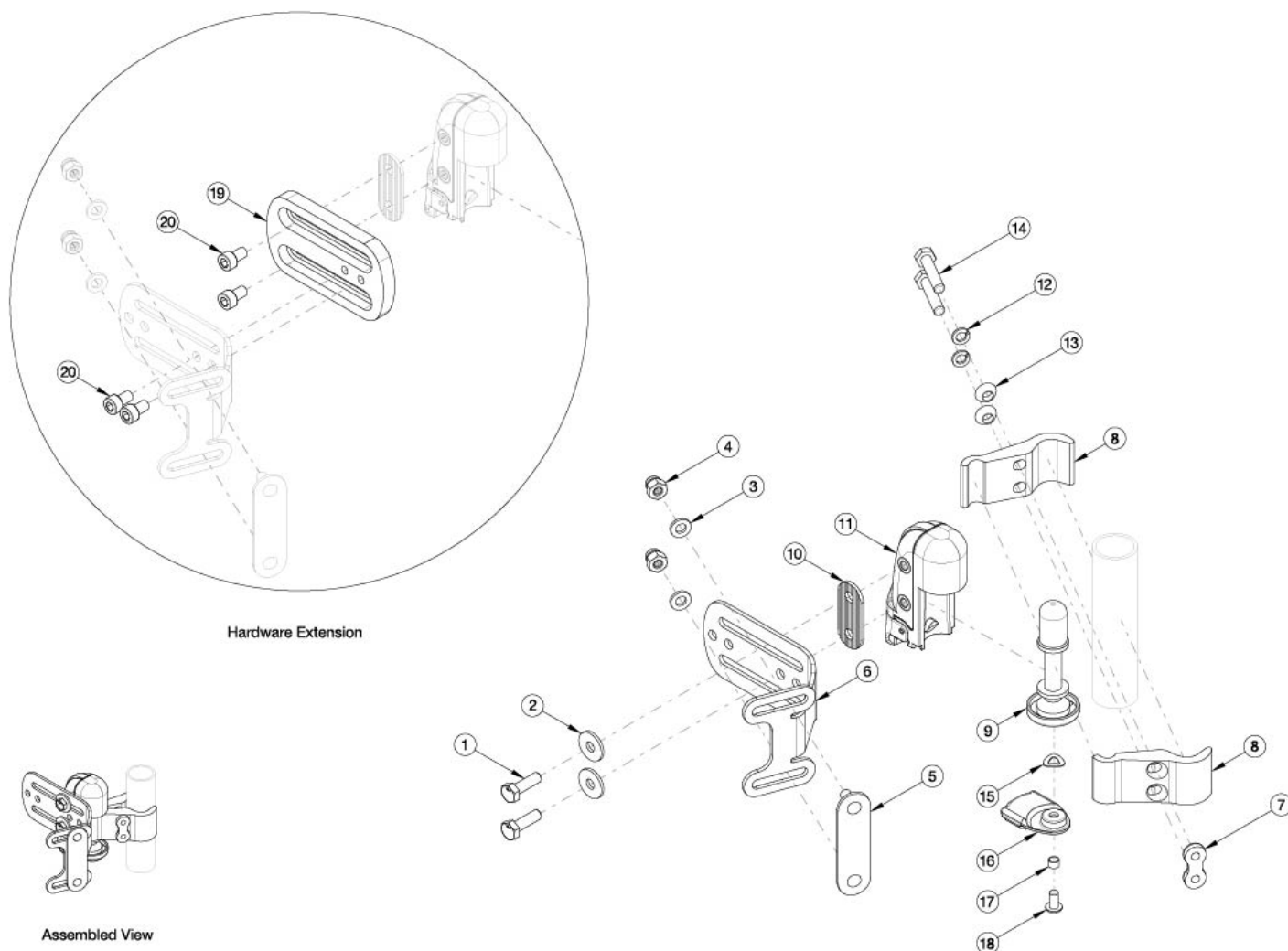

Item Number	Part Number	Description	Quantity	Retail
1-18	122277	Ped Hardware Retrofit Kit, PAIR, Pediatric Back	1	\$485.00
19-20	122278	Pediatric DEX HDW Extension	1	\$116.88
1-2, 10-11b	122279	Ped Housing and Hardware Kit, Pair, Pediatric Back	1	\$77.08
1-2, 10-11a	122280	Ped Housing and Hardware Kit, Left, Pediatric Back	1	\$38.54
1-2, 10, 11b	122281	Ped Housing and Hardware Kit, Right, Pediatric Back	1	\$38.54
7-8, 12-14	122282	PED Clamp Assembly, Hardware Included, Pair, Pediatric Back	1	\$281.52
7-8, 12-14	122283	PED Clamp Assembly, Hardware Included, Each, Pediatric Back	2	\$140.76
3-6	122284	Ped Back Shell Mount and Hardware Kit, Pair, Pediatric Back	1	\$134.32

Item Number	Part Number	Description	Quantity	Retail
3-6	122285	Ped Back Shell Mount and Hardware Kit, Each, Pediatric Back	2	\$67.16
9, 15-18	122286	Ped Mount Post, Latch, and Hardware Kit, Pair, Pediatric Back	1	\$113.68
9, 15-18	122287	Ped Mount Post, Latch, and Hardware Kit, Each, Pediatric Back	2	\$56.84
1	101823	M6 X 20 Hhcs Blk Zn	4	\$1.17
2	101853	M6 Washer Blk Zn	4	\$0.60
3	100746	M6 Flat Washer Blk Zn	4	\$1.77
4	100558	M6 Nyl Ins Domed Cap L/N Zc	4	\$1.17
5	115568	Back Mount, Stud Plate, Fxt	2	\$7.28
6	117359	Pediatric Back Mount Bracket	2	\$54.00
7	005476	Nut, Dual, Sr.25, M6, .60Sp	2	\$39.96
8	005472	Axiom Back Mount Clamp	4	\$34.25
9	117907	Mount Post, Locking, Pediatric Backrest	2	\$42.00
10	115625	Dual Washer, Vee Groove, Backrest	2	\$11.00
11a	122968	Axiom Back Mount Housing Sub Assembly, Left	1	\$24.00
11b	122969	Axiom Back Mount Housing Sub Assembly, Right	1	\$24.00
12	101835	M6 Lock Washer Blk Zn	4	\$0.60
13	003810	Washer, Spher, 10Mm	4	\$11.42
14a	100860	M6 X 35 Hhcs P/Thd Zc	2	\$1.77
14b	101824	M6 X 30 Hhcs Blk Zn	2	\$2.36
15	117906	Washer, M6, Wave, Din137B, Black Oxide	2	\$1.07
16	117358	Latch, Mount Post, Pediatric Backrest	2	\$12.00
18	100669	M5 x 10 BTN SHCS BLKZC w/Patch	2	\$1.77
19	117360	Pediatric Back Mount Bracket, Extension	8	\$49.00
20	101179	M6 X 10 Shcs Blk Zc W/Patch	2	\$2.36