


Rogue2 Amputee Camber Mount

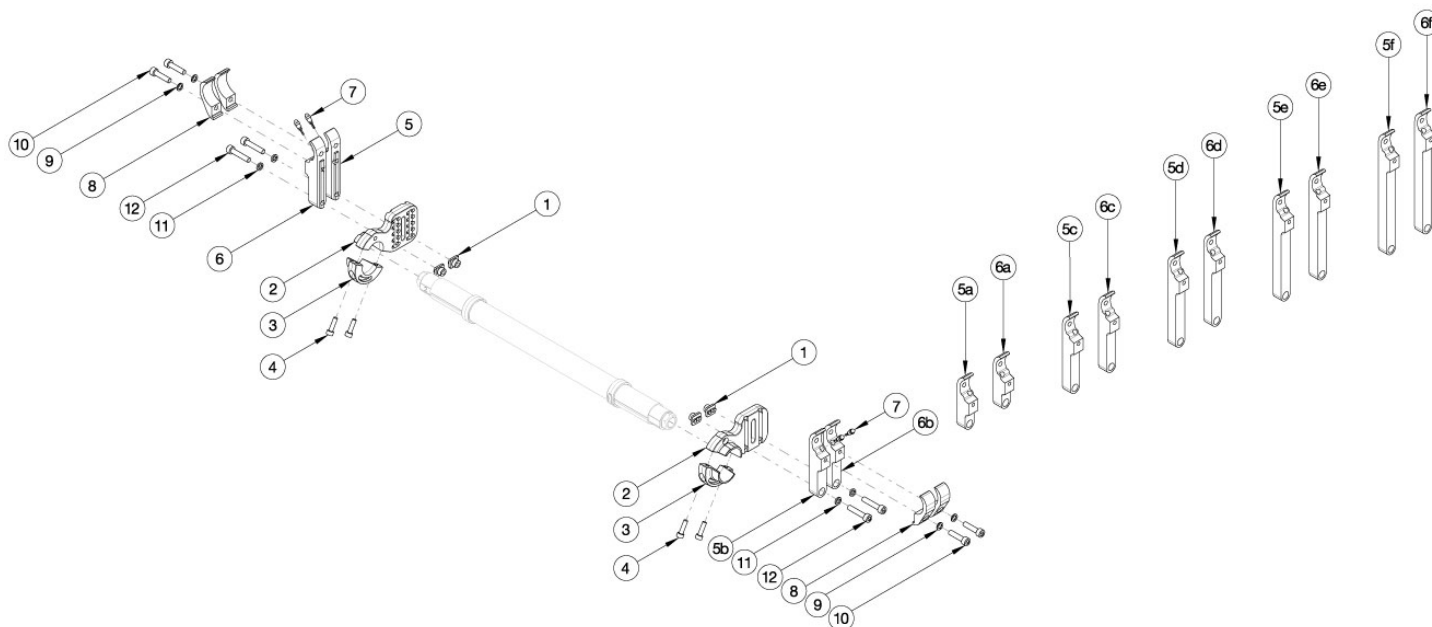

Please contact Customer Service if "Rear Inner Tower Clamps" are needed. If ordering "Rear Inner Tower Clamps", detent buttons (001788) are required. Used when center of gravity is between -1.5" and 0.25". If One Arm Drive is present, refer to the corresponding One Arm Drive parts manual page for the correct towers.

Item Number	Part Number	Description	Quantity
1, 2a, 3, 4, 8a-12	117504	Rear Tower Assembly Amputee, Black, Right, Rogue2 / Rogue XP	1
1, 2a, 3, 4, 8b-12	117504_ _ _	Rear Tower Assembly Amputee, Anodized/Color Accent, Right, Rogue2 / Rogue XP	1
1, 2b, 3, 4, 8a-12	117505	Rear Tower Assembly Amputee, Black, Left, Rogue2 / Rogue XP	1
1, 2b, 3, 4, 8b-12	117505_ _ _	Rear Tower Assembly Amputee, Anodized/Color Accent, Left, Rogue2 / Rogue XP	1
1, 2a, 2b, 3, 4, 8a-12	117506	Rear Tower Assembly Amputee, Black, Pair, Rogue2 / Rogue XP	1
1, 2a, 2b, 3, 4, 8b-12	117506_ _ _	Rear Tower Assembly Amputee, Anodized/Color Accent, Pair, Rogue2 / Rogue XP	1
1	114014	Rear Tower Square Nut	2
2a	123008	Amputee Camber Mount Clamp Base Right	1
2b	123009	Amputee Camber Mount Clamp Base Left	1
3	114021	Amputee Camber Mount Clamp, Lower	1
5a	114004	Rear Tower Inner Index Clamp 2.125", Black, Right	2
5a	114004_ _ _	Rear Tower Inner Index Clamp 2.125", Anodized/Color Accent, Right	2
5b	114005	Rear Tower Inner Index Clamp 2.875", Black, Right	2

Item Number	Part Number	Description	Quantity
5b	114005_ _ _	Rear Tower Inner Index Clamp 2.875", Anodized/Color Accent, Right	2
5c	114006	Rear Tower Inner Index Clamp 3.625, Black, Right	2
5c	114006_ _ _	Rear Tower Inner Index Clamp 3.625, Anodized/Color Accent, Right	2
5d	114007	Rear Tower Inner Index Clamp 4.500", Black, Right	2
5d	114007_ _ _	Rear Tower Inner Index Clamp 4.500", Anodized/Color Accent, Right	2
5e	114008	Rear Tower Inner Index Clamp 5.375", Black, Right	2
5e	114008_ _ _	Rear Tower Inner Index Clamp 5.375", Anodized/Color Accent, Right	2
5f	114009	Rear Tower Inner Index Clamp 6.250", Black, Right	2
5f	114009_ _ _	Rear Tower Inner Index Clamp 6.250", Anodized/Color Accent, Right	2
6a	114034	Rear Tower Inner Index Clamp 2.125", Black, Left	2
6a	114034_ _ _	Rear Tower Inner Index Clamp 2.125", Anodized/Color Accent, Left	2
6b	114035	Rear Tower Inner Index Clamp 2.875", Black, Left	2
6b	114035_ _ _	Rear Tower Inner Index Clamp 2.875", Anodized/Color Accent, Left	2
6c	114036	Rear Tower Inner Index Clamp 3.625, Black, Left	2
6c	114036_ _ _	Rear Tower Inner Index Clamp 3.625, Anodized/Color Accent, Left	2
6d	114037	Rear Tower Inner Index Clamp 4.500", Black, Left	2
6d	114037_ _ _	Rear Tower Inner Index Clamp 4.500", Anodized/Color Accent, Left	2
6e	114038	Rear Tower Inner Index Clamp 5.375", Black, Left	2
6e	114038_ _ _	Rear Tower Inner Index Clamp 5.375", Anodized/Color Accent, Left	2
6f	114039	Rear Tower Inner Index Clamp 6.250", Black, Left	2
6f	114039_ _ _	Rear Tower Inner Index Clamp 6.250", Anodized/Color Accent, Left	2
7	001788	Button Assembly	2
8a	114003	Rear Tower Outer Clamp, Black	4
8b	114003_ _ _	Rear Tower Outer Clamp, Anodized//Color Accent	4
9	102326	Flat Washer ID .254" x OD .396" Thickness .047" Zinc	2
10	100780	M6 x 25 SHCS ZC	2
11	102810	M6 Lockwasher, Hi-Collar, ZC	2

Item Number	Part Number	Description	Quantity
12	102627	M6 X 30 SHCS BLK ZC	2