


## Focus CR Caster Arm

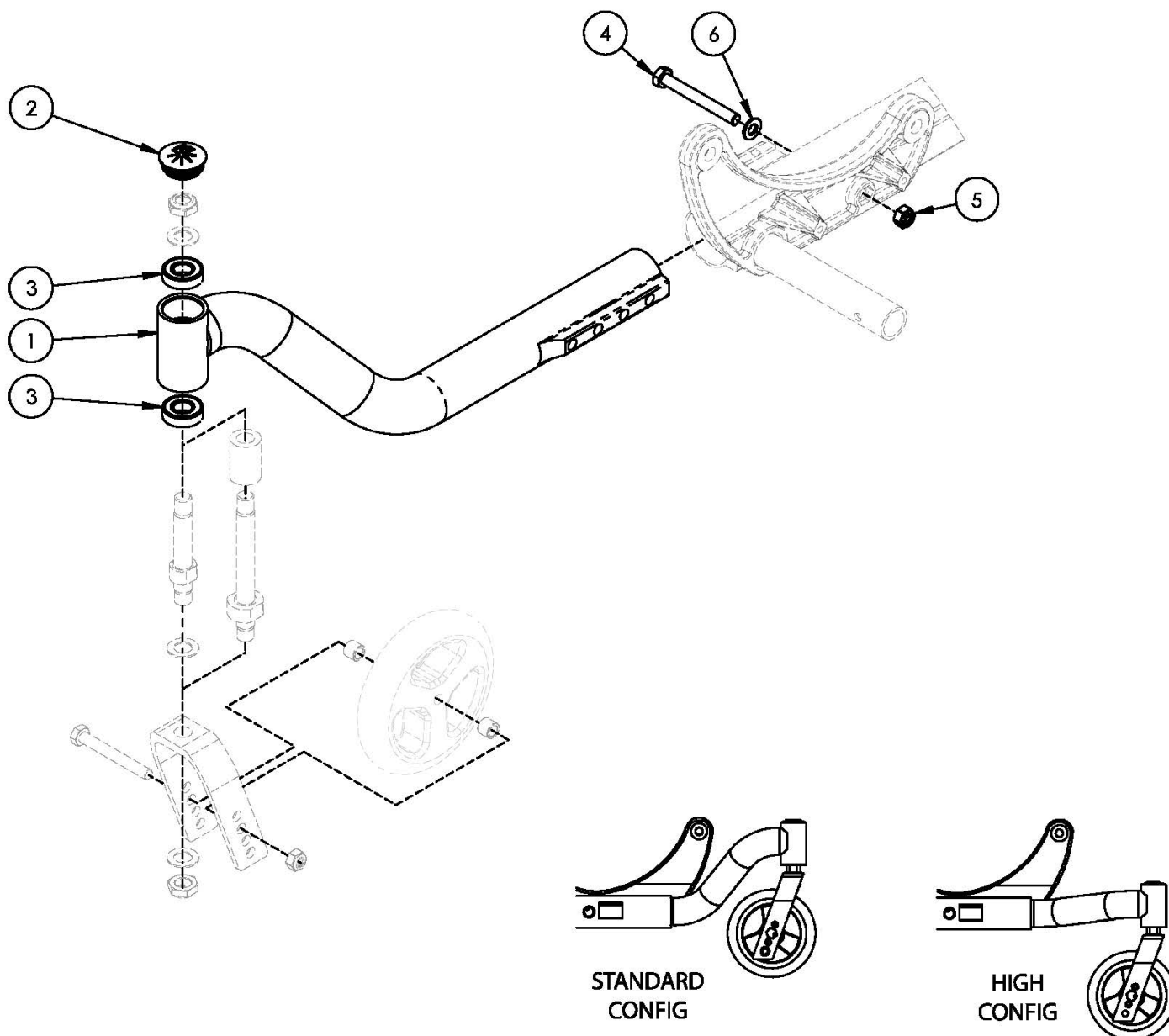

| Item Number | Part Number | Description                                      | Quantity | Retail   |
|-------------|-------------|--------------------------------------------------|----------|----------|
| 1a, 2, 3    | 501067_ _ _ | Frame Assembly Focus Caster Arm Right _ _ _      | 1        | \$249.53 |
| 1b, 2, 3    | 501068_ _ _ | Frame Assembly Focus Caster Arm Left _ _ _       | 1        | \$249.53 |
| 1c, 2, 3    | 501426_ _ _ | Frame Assembly Focus Caster Arm High Right _ _ _ | 1        | \$249.53 |
| 1d, 2, 3    | 501427_ _ _ | Frame Assembly Focus Caster Arm High Left _ _ _  | 1        | \$249.53 |
| 2           | 102005      | Ki Caster Housing Cap                            | 1        | \$1.17   |

| Item Number | Part Number | Description                | Quantity | Retail |
|-------------|-------------|----------------------------|----------|--------|
| 3           | 100902      | Bearing 1 1/8" x 1/2" R8RS | 2        | \$6.30 |
| 4           | 101830      | M8 x 70 HHCS Blk Zn        | 1        | \$2.63 |
| 5           | 100659      | M8 Nylock Nut Blk Zn       | 1        | \$1.05 |
| 6           | 101858      | Washer Flat M8 Blk Zn      | 1        | \$0.53 |

| Matte Finish |                     |
|--------------|---------------------|
| Code         | Color Name          |
| CGM          | Charcoal Gray       |
| EGM          | Emerald Green       |
| GGM          | Graphite Gray*      |
| IBM          | Iridescent Burgundy |
| INM          | Iridescent Navy     |
| AGM          | Military Green**    |
| RRM          | Retro Red           |
| RBM          | Royal Blue          |
| SBM          | Shadow Black        |
| SWM          | Smooth White        |

\* Discontinued Color

\*\* Formerly called Army Green

| Gloss Finish |                        |
|--------------|------------------------|
| Code         | Color Name             |
| BLK          | Black                  |
| BLU          | Blue                   |
| SKY          | Blue Skies             |
| BRO          | Bright Orange*         |
| PNK          | Bubblegum Pink         |
| BUR          | Burgundy*              |
| CBL          | Candy Blue             |
| PUR          | Candy Purple           |
| RED          | Candy Red              |
| COP          | Copper                 |
| DPR          | Deep Purple*           |
| EBL          | Electric Blue*         |
| GRN          | Forest Green*          |
| FUS          | Fuchsia*               |
| GSM          | Granny Smith           |
| MTF          | Metallic Fuchsia*      |
| MOR          | Metallic Orange        |
| ORG          | Orange*                |
| SAP          | Sapphire*              |
| SIL          | Silver                 |
| TEA          | Teal                   |
| YLW          | Yellow                 |
| FLM          | Graphic Image Transfer |