



TRILOGY VENT MOUNT



Please read these instructions carefully before beginning the assembly. Failure to understand and follow assembly instructions may result in injury to technician and/or end user and may void the warranty. If you have any questions call Sunrise Medical Technical Support at +1 (800) 333-4000.

Parts included:

1. Vertical vent tray with mounting hardware
2. Trilogy dovetail mount adapter
3. Trilogy EVO mount hardware (3 X HHCS and Nylock Nuts)

Required tools:

1. 13 mm wrench or ratchet
2. 7/64 Allen key for Trilogy100 Style Vents (1X)



1

A. TRILOGY EVO OR SIMILAR

1 2

Mounting the Trilogy EVO vent requires the use of a wheelchair mounting kit (Philips Respironics P/N 1134633) (Fig 1) which can be purchased directly from Philips Respironics.

1. Attach Respironics mounting bracket to the Sunrise provided vent tray using 3 HHCS and Nylock Nuts. (Fig 2)
2. Tighten nuts until snug.
3. Attach Respironics mounting plate to the bottom of the Trilogy ventilator per the manufacturer's recommendations.

B. TRILOGY 100 OR SIMILAR

3 4

Mounting the Trilogy 100 or similar vent requires the use of the four threaded mounting holes located on the back of the ventilator. (Fig 3)

1. Attach the Trilogy dovetail mount adapter to the back of the Trilogy vent. If the vent is equipped with a Trilogy Roll Stand adapter, the dovetail mount adapter should be mounted between the roll stand adapter and the vent. (Fig 4)

Sunrise Vent Tray



Philips Respironics Wheel Chair Mounting Kit

2

3



4



DEALER/TECHNICIAN WARNING

Attention dealers and qualified technicians, do not operate or service this wheelchair without first reading the owners manual. If you do not understand the instructions and warnings in the owners manual please contact the Sunrise Medical Technical Service Department before operating and/or servicing the Quickie/Zippie wheelchair. Failure to do so may result in damage and/or injury. Find more information and important warnings see the wheelchair owner's manual or at: www.SunriseMedical.com.

Sunrise Medical 2842 N. Business Park Ave Fresno, CA 93727 USA

Customer service: +1(800) 333-4000
or visit www.SunriseMedical.com



© 2021 Sunrise Medical (US) LLC
08/2021 255610 Rev.A Page 1 of 1